

Nutripore™

Blood Collection Card for Vitamin D Analysis

IFU

Instructions For Use

For healthcare professionals

Contents

- 1 x Nutripore Blood Collection Card for Vitamin D Analysis in foil pouch
- 1 x Desiccant (moisture absorbing sachet)
- 4 x Extra QR code labels (for laboratory records if needed)
- 1 x Return address label

Important

⚠ Caution

Do NOT touch surface of circles on collection card.

Information and Support

Visit www.trajannutrition.com or contact info@trajannutrition.com

Specifications are subject to change without notice.

Instructions

Read all the instructions first before commencing.


1. Please wash hands in warm water. Let your hands dry completely before putting on gloves. Nitrile disposable gloves are recommended.
2. Open foil zip-lock pouch and remove Nutripore Blood Collection Card (do not remove desiccant). Reseal and retain for later.
3. Unfold Nutripore Blood Collection Card, **be careful not to touch the surface of the circles.**
4. Unlock lancet (not supplied).
5. To prepare the finger for blood collection, warm the chosen finger by massaging it. Use alcohol swab (not supplied) to wipe area on fingertip. Allow to dry.
6. We recommend to use the non-dominant hand for the finger prick. Puncture the pulpy surface of the last segment of the finger, towards the side. Avoid using little finger.
7. Position lancet firmly on side of fingertip and press the release button on the side to activate. Discard lancet appropriately.
8. Increase blood flow by holding puncture site downward and gently applying pressure to surrounding area.



9. Carefully place the droplet of blood in each circle, aiming to fill each circle completely (this may require 1-2 drops of blood). **Avoid touching the circle with skin or smearing the blood.**
10. Apply self adhesive bandage (not supplied) provided to cover the finger until it is healed.
11. The specimen must be labelled immediately after the sample is collected. On the Patient ID Form panel; write the study ID (if required) and/or affix a QR code label, write date of birth and date of collection.
12. Place Nutripore Blood Collection Card open on its side, being careful not to touch the blood drops. Air dry for 1 hour.
13. Once dry, fold Nutripore Blood Collection Card over. **Make sure to place the collection circles on the inside of the cover.**
14. Place Nutripore Blood Collection Card in foil pouch with desiccant. **Make sure to seal securely.**
15. Return in the mail to Trajan Nutrition laboratory or your laboratory of choice:

Trajan Nutrition
c/o SAHMRI
George Street delivery dock
Adelaide SA 5000
Australia

TRAJAN
nutrition

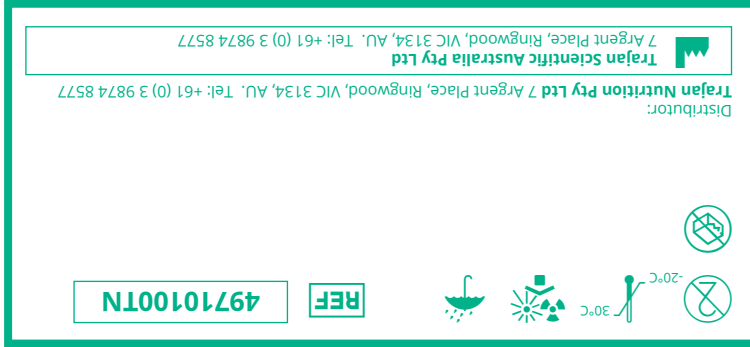

Nutripore™

Blood Collection Card for Vitamin D Analysis



Instructions For Use
For healthcare professionals

TRAJAN
nutrition

Introduction

The Nutripore™ Blood Collection Card for Vitamin D Analysis is intended for the collection of a blood sample, using a Nutripore Blood Collection Card for vitamin D analysis. The finger prick method is used to produce blood droplets that are directly transferred to the collection card and analysed in a laboratory.

For Research Use Only. Not for Therapeutic or Diagnostic Use.